

## GeneProof Herpes Simplex Virus (HSV-1/2) PCR Kit



*In vitro* diagnostic medical device

The kit has been manufactured according to EC Directive 98/79/EC as an *in vitro* diagnostic medical device and it has been designed for professional use in specialized clinical and research laboratories.

### KIT CONTENT

REF	ISIN Version IS included in the MasterMix			ISEX Version IS supplied in a separate tube Nucleic acid isolation and PCR inhibition control		
	HSV/ISIN/025 25 rxn	HSV/ISIN/050 50 rxn	HSV/ISIN/100 100 rxn	HSV/ISEX/025 25 rxn	HSV/ISEX/050 50 rxn	HSV/ISEX/100 100 rxn
<b>MasterMix</b> HSV	1x750 µl	2x750 µl	4x750 µl	1x750 µl	2x750 µl	4x750 µl
<b>CALIBRATOR</b> HSV 10 <sup>4</sup> copy/µl	1x200 µl	1x200 µl	1x200 µl	1x200 µl	1x200 µl	1x200 µl
<b>CALIBRATOR</b> HSV 10 <sup>3</sup> copy/µl	1x200 µl	1x200 µl	1x200 µl	1x200 µl	1x200 µl	1x200 µl
<b>CALIBRATOR</b> HSV 10 <sup>2</sup> copy/µl	1x200 µl	1x200 µl	1x200 µl	1x200 µl	1x200 µl	1x200 µl
<b>CALIBRATOR</b> HSV 10 <sup>1</sup> copy/µl	1x200 µl	1x200 µl	1x200 µl	1x200 µl	1x200 µl	1x200 µl
<b>Internal Standard</b> HSV	-	-	-	1x1000 µl	1x1000 µl	2x1000 µl

### STORAGE AND TRANSPORTATION CONDITIONS

The kits should be transported and stored at temperatures between -85 °C and -10 °C. The kit will remain stable at least until the expiry date printed on the package, if the storage temperature is kept. Repeated freezing and thawing of the kit components may result in lower detection quality.

### TECHNICAL SPECIFICATION

<b>Target sequence</b>	DNA conservative section of a single-copy gene encoding the B glycoprotein B (gB)
<b>Specificity</b>	Herpes simplex virus type 1 (HSV-1) Herpes simplex virus type 2 (HSV-2)
<b>Sensitivity (LOD)</b>	for HSV-1 reaches 0.764 copies/µl with the probability of 95 % for HSV-2 reaches 1.309 copies/µl with the probability of 95 %
<b>Accuracy of measurement</b>	for HSV-1 within the range of 10 <sup>4</sup> -0.764 copies/µl the detection accuracy is 0.5 log for HSV-2 within the range of 10 <sup>4</sup> -1.309 copies/µl the detection accuracy is 0.5 log
<b>Linear range of measurement</b>	for HSV-1 within the range of 10 <sup>7</sup> -0.764 copies/µl for HSV-2 within the range of 10 <sup>7</sup> -1.309 copies/µl
<b>Sample types</b>	whole blood in EDTA, plasma, cerebrospinal fluid, swabs, urine
<b>Reporting units</b>	copies/ml
<b>Quality Control</b>	regularly tested by QCMD and Instand e.V. External Quality Assessment Panels



## METHOD PRINCIPLES

The PCR kit is designed for detection and differentiation of both types of Herpes simplex viruses (HSV-1 and HSV-2) by the real-time Polymerase Chain Reaction (multiplex PCR) method. The HSV detection is based on the amplification of a specific conservative DNA sequence of a single-copy gene encoding glycoprotein B (gB) and measuring the amplification product concentration using the PCR process and fluorescence labelled probes. HSV-1 presence is indicated by FAM fluorophore fluorescence growth and the presence of HSV-2 is indicated by Cy5 fluorophore fluorescence growth. An Internal Standard (IS) is included in the reaction mix, controlling the possible inhibition of the PCR reaction (ISIN version) and possibly also the DNA extraction process quality (ISEX version). IS positive amplification is detected in the HEX fluorophore fluorescence channel. The detection kit takes advantage of the "hot start" technology, minimizing non-specific reactions and assuring maximum sensitivity. Ready to Use MasterMix contains uracil-DNAGlycosylase (UDG), eliminating possible contamination of the PCR reaction by amplification products. The kit performs very sensitive HSV-1 and HSV-2 detection in clinical material (EDTA whole blood, plasma, cerebrospinal fluid, swab, urine). The kit is designed for *in vitro* diagnostics and provides qualitative and quantitative detection.


### ISIN version

Internal Standard is included in the MasterMix tube. This PCR kit version enables PCR inhibition control.

### ISEX version

Internal Standard is provided as independent item within the package. This PCR kit version enables both PCR inhibition control and nucleic acid purification process efficiency control.

## MICROBIOLOGICAL DNA DETECTION TECHNOLOGY

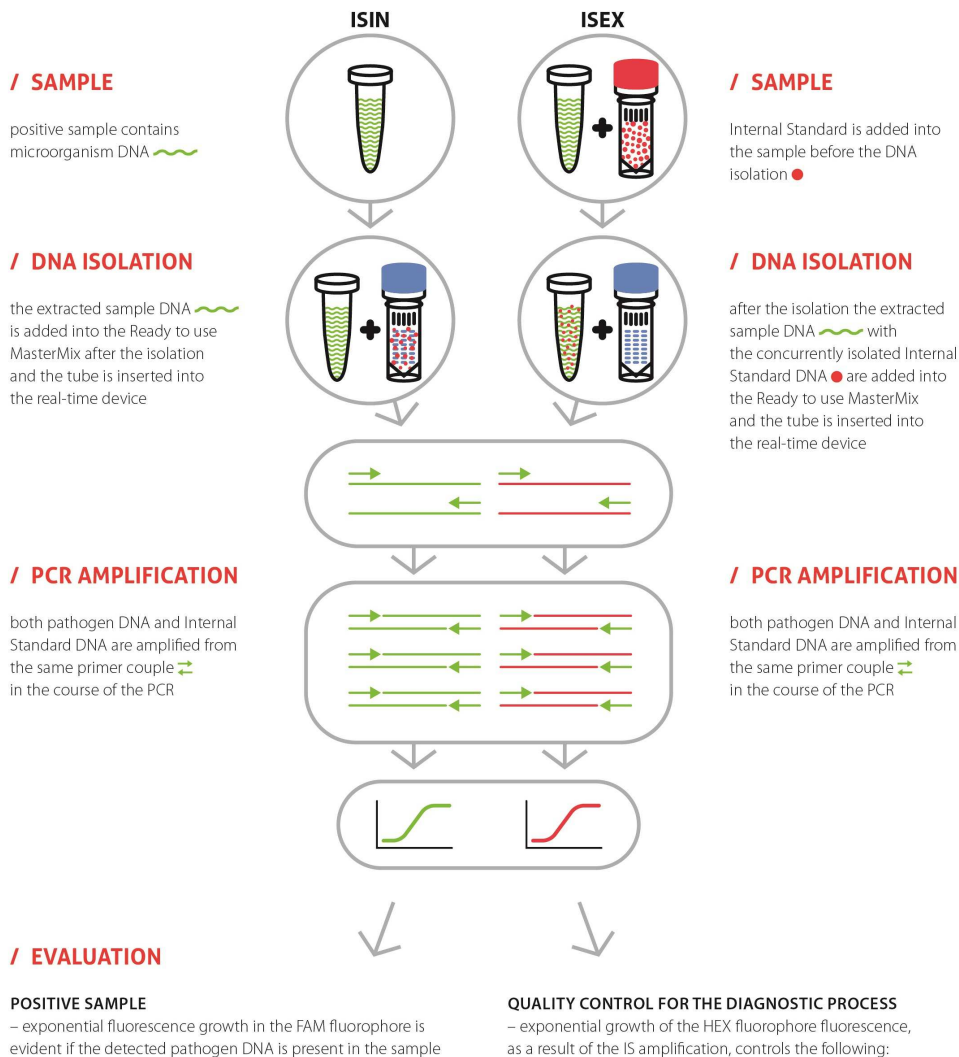

## USER MANUAL

### SAMPLING AND SAMPLE STORAGE

Sampling of all sample types (plasma, cerebrospinal fluid, swabs, urine), except for blood, should be performed into sterile tubes without any transportation media and the samples should be transported within 12 hours at the temperature between +2 °C and +8 °C. Non-coagulating peripheral blood should be sampled into EDTA and transported to the laboratory at the temperature between +2 and +8 °C within 24 hours. In case of longer storage keep all samples frozen at the temperature between -85 and -10 °C.

### NUCLEIC ACID PURIFICATION

Nucleic acid isolation should be performed by isolation kits available at the market according to protocols for the particular clinical material isolation. The manufacturer recommends the following isolation kits:

GeneProof PathogenFree DNA Isolation Kit  
croBEE NA16 Nucleid Acid Extraction System

When using the ISEX versions of the PCR kits the IS should be added directly into the sample at the beginning of the isolation process so that in the end 1 µl of the resulting elution volume contains 0.1 µl of the IS:

Elution volume	25 µl	50 µl	100 µl	200 µl
Internal Standard	2,5 µl	5 µl	10 µl	20 µl

### PCR SETUP

1. Add 30 µl of MasterMix into PCR tubes.

2. Add 10 µl of the isolated nucleic acid sample or 10 µl of Positive Control into the individual PCR tubes. The final reaction mix volume will be 40 µl.

*It is necessary to keep all components at +2°C to +8°C during the PCR preparation.*

3. Close the tubes, centrifuge shortly, insert them into the device and let them amplify according to the following PCR profile.

*Be very careful when handling the Positive Control or the clinical material, incorrect handling could result in contamination and the consequent impairment of the kit components or the MasterMix! The manufacturer is not responsible for the kit impairment due to incorrect handling.*

### AMPLIFICATION PROFILE

Step	Temperature	Time	Data collection	Cycles
1. Hold	37 °C	2 min		1
2. Hold	95 °C	10 min		1
3. PCR	95 °C	5 s		
	60 °C	40 s	FAM+HEX+Cy5	45
	72 °C	20 s		

### VALIDATED INSTRUMENTS

GeneProof PCR kits are designed for use with real-time devices from various manufacturers. This PCR kit has been validated with the following devices:

Applied Biosystems 7500 Real-Time PCR System

AriaMx Real-Time PCR System

Dx/CFX96™ Real-Time PCR Detection System

LightCycler® 480

LineGene 9600\*

Mx3000P/3005P QPCR System

Rotor-Gene 3000\*, Rotor-Gene Q\*



















SLAN® Real-Time PCR System

\* Validation applies to a device model providing detection in the following channels: FAM, HEX and Cy5.

GeneProof diagnostic kits are continually validated with various types of devices. Please request the current list at support@geneproof.com.



## CLINICAL SAMPLE ANALYSIS EVALUATION

Channel FAM	Channel Cy5	Channel HEX	Result	Interpretation
			Valid	HSV-1 positive
			Valid	HSV-2 positive
			Valid	HSV-1+HSV-2 positive
			Valid	negative
			Invalid	
			Invalid	

## QUANTITATIVE DETECTION EVALUATION

Use the following formula to calculate the virus concentration in copies/ml while taking into account the volume of material entering the isolation:

$$\text{copy/ml} = \frac{\text{SC} \times \text{EV}}{\text{IV}}$$

SC - Sample Concentration (copy/μl)

EV - Elution Volume (μl)

IV - Isolation Volume (ml)

You can use the calculator for pathogen concentration conversion at [www.geneproof.com](http://www.geneproof.com) to make the calculation easier.

### WARNING

A single valid package insert for a specific kit is included in the package or to be requested for the particular lot from the manufacturer. The kit should be disposed of after use according to the current legal regulations considering the fact that the kit doesn't contain any dangerous, infectious or toxic components that would be subject to special safety regulations, and the packaging materials are made of paper and polypropylene. If you have any questions please contact our Customer Service.

#### Customer care and technical support

Tel.: +420543211679  
 Fax: +420516770824  
 email: [support@geneproof.com](mailto:support@geneproof.com)

#### Orders

Tel.: +420543211679  
 Fax: +420516770824  
 email: [sales@geneproof.com](mailto:sales@geneproof.com)

